

## HEADQUARTERS

Apex Medical Corp.  
No. 9, Min Sheng St., Tu-Cheng,  
New Taipei City, 23679, Taiwan  
T +886 2 2268 5568  
F +886 2 2268 6525  
marketing@apexmedicalcorp.com

## PAN-ASIA

### TAIWAN

Apex Medical Corp.  
No. 9, Min Sheng St., Tu-Cheng,  
New Taipei City, 23679, Taiwan  
T +886 2 2268 5600 /0800 054 133  
F +886 2 2268 6526  
customer@apexmedicalcorp.com

### STURDY INDUSTRIAL

168, Sec.1, Zhongxing Road, Wugu District,  
New Taipei City 24872, Taiwan  
T +886 2 2981 5245  
F +886 2 2984 8208  
foreign@sturdy.com.tw  
www.sturdy.com.tw

### CHINA

Apex Medical (Kunshan) Corp.  
No. 1368, ZiZhu Road, Kunshan,  
Jiang Su Province, China  
T +86 512 8617 1660 Ext 1150  
F +86 512 8617 8663  
customer.cn@apexmedicalcorp.com

### INDIA

Apex Medicalcorp India Pvt. Ltd.  
303, Third Floor Block 4A DLF  
Corporate Park Phase-III,  
Gurgaon, Haryana  
T +91 11 6654 4138  
F +91 11 6654 4052  
enquiries.india@apexmedicalcorp.com

### THAILAND

Apex Medical Thailand  
65/100 Chamnanphenjati Business Center Bldg.  
11/F Rama 9 Road, Huaykwang  
Bangkok 10320  
T +662 948 4848  
F +662 948 4170  
sales.thai@apexmedicalcorp.com

### MIDDLE EAST • AFRICA

29, Misr Liltameer Building,  
Stree#10, AREA#8, Sheraton Building  
Heliopolis Cairo, Egypt.  
T +202 2 2671303  
F +202 2 2678649  
raafat.wahby@apexmedicalcorp.com

## PAN-EUROPE

### SPAIN

Apex Medical S.L.  
Elcano 9, 6ª planta 48008  
Bilbao. Vizcaya. Spain  
T +34 94 4706408  
F +34 94 4706409  
info@apexmedical.es

### UK

Apex Medical Limited  
Unit 33, Great Western Business Park  
McKenzie Way, Worcester ,WR4 9GN  
United Kingdom  
T +44 (0) 1905 774 695  
sales@apexmedicalcorp.co.uk

### FRANCE

Apex Medical France  
4 Boulevard de la Chanterie  
49124 Saint-Barthelemy d'Anjou  
France  
T +33 2 72 79 27 13  
F +33 2 85 73 07 67  
info.france@apexmedicalcorp.com

### GERMANY

SLK Vertriebsgesellschaft GmbH  
Lindenhorster Straße 38-40 44147  
Dortmund  
T +49 (0)231-925360-175  
F +49 (0)231-925360-29  
info@slk-gmbh.de  
www.stk-gmbh.de

## AMERICAS

### USA

Apex Medical USA  
927 Mariner Street,  
Brea, CA 92821, USA  
T +1 714 671 3818  
F +1 714 494 8173  
sales.usa@apexmedicalcorp.com

Distributed by



# Domus 4


## Alternating Pressure Redistribution System

- Suitable for patients with medium to high risk of pressure injury
- Simple-to-use weight setting function
- CPR knob for quick deflation & maximum security
- Heel relief function for optimal heel protection



Care for a Healthy Life

**APEX**

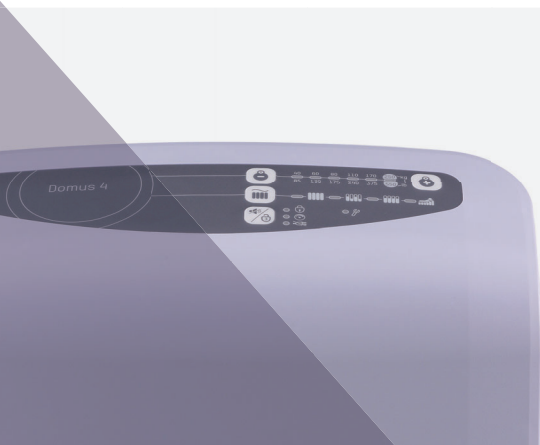

# Domus 4

## Alternating Pressure Redistribution System



**Cell-on-Cell Design**  
The bottom cells prevent the patient from bottoming out in case of power failure



**Micro Low Air Loss**  
Helps wick away moisture from the surface to keep patient's skin less susceptible to injuries



**Heel Relief Function**  
Individual detachable air cells prevent pressure injuries on the heels




**Visual and Audible Alarm**

- Low Pressure
- Power Failure
- Tech Support



**CPR Knob**  
Designed for quick mattress deflation in case of emergency and prevents accidental triggering

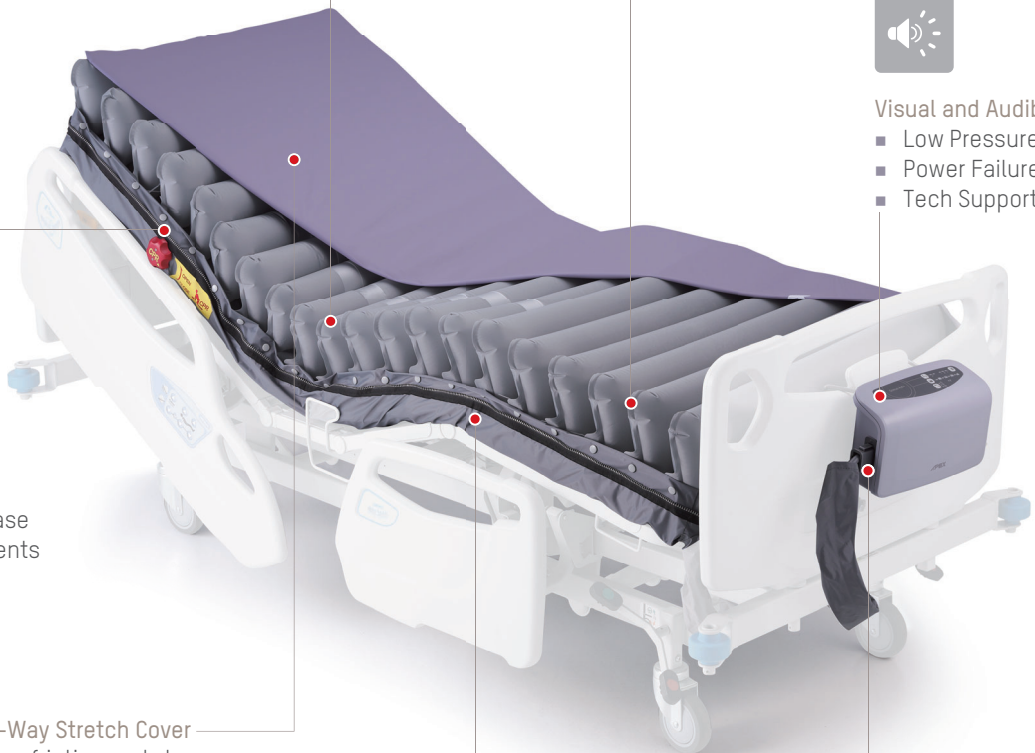

**4-Way Stretch Cover**  
Low friction and shear cover with high moisture-vapour transmission rate for better microclimate management



Biocompatibility



Vapour Permeable



Water Resistance



Flame Resistance




**Enhanced Cross-Infection Control**  
Made with all-around-zipper and welded seams for easy removal and prevents fluid ingress



**Quick Connector with Cap**  
Facilitates system installation and seals the mattress for patient transportation up to 8 hours

# Pressure Relief During Alternating

The Domus 4 is designed to maximize patient's safety and offers multiple therapy modes that are capable of fulfilling various clinical needs for better patient care.

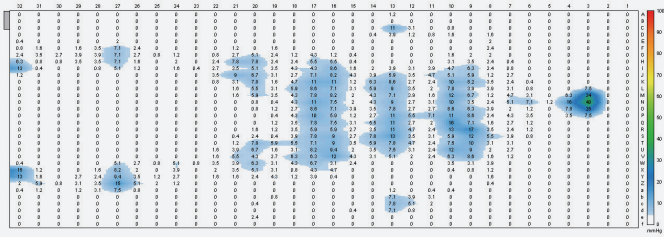

## 5 Modes to Fulfill Individual Needs

- Max Firm
- Alternating
- Continuous Low Pressure (CLP)
- Seat Inflation w/ Alternating
- Seat Inflation w/ CLP



### Alternating

- Alternately inflates and deflates air cells allowing pressure redistribution and encourage tissue reperfusion



### Constant Low Pressure (CLP)

- Provides a constant static surface in lower pressure with a greater contact area for patients to be immersed and enveloped better



### Max Firm

- A static firm surface for nursing procedures. It will automatically return to alternating mode after 20 minutes



### Seat Inflation

- Adjust the pressure to suit the patient in a sitting position. It can be used with alternating or CLP mode

## Specifications

Model	Domus 4	
Pump	Dimension	29 x18.5 x 12.6 cm
	Weight	2.2 kg
	Case Material	Flame Retardant ABS
	Supply Voltage	AC 220 – 240, 50 Hz
	Operating Cycle	10 mins
Mattress	8" Replacement	
	Dimension	200 x 80/85/90 x 20.3 cm
	Cell Height	8" x 20 Cells
	Weight	6.9 kg
	Cover Material	4-Way Stretch
	Cell Material	Nylon TPU
	Maximum Patient Weight	250 kg
Flame Retardant Standards	EN 591-1; EN 591-2	

Specifications are subject to change for improvement without notice