

	Product Specification: TF-18	PS-0007-2020
	Sterile First Aid Dressings	
	Version: 1.03	Page 1 of 3

Product Description: Sterile First Aid Dressings

Product Code:

Product Code	Description
--------------	-------------

30ABDT01	Blue Dot Sterile No.1 Ambulance Dressing
30ABDT02	Blue Dot No.2 Ambulance Dressing
30ABDT03	Blue Dot No.3 Ambulance Dressing
30ABDT04	Blue Dot No.4 Ambulance Dressing
30BDFD35	Blue Dot Sterile Finger Bandage 3.5 x3.5cm
30BDFD55	Blue Dot Sterile Finger Bandage 5 x 5cm
30FBDT03	Blue Dot Sterile No.3 Extra Large Dressing
30FBDT07	Blue Dot Sterile No.7 Small Dressing
30FBDT08	Blue Dot Sterile No.8 Medium Dressing
30FBDT09	Blue Dot Sterile No.9 Large Dressing
30FBDT16	Blue Dot Sterile No.16 Eye Pad Dressing
30FMD120	Blue Dot Sterile Medium Dressing
30FLG180	Blue Dot Sterile Large Dressing
30BCRE03	Standard Dressing Boxed No.3 Extra Large Dressing
30BCRE08	Standard Dressing Boxed No.8 Sterile Medium Dressing
30BCRE08-10	Standard Dressing Boxed No.8 Sterile Medium Dressing (Pack of 10)
30BCRE09	Standard Dressing Boxed No.9 Sterile Large Dressing
30BCRE16	Boxed No.16 Eye Pad Sterile Dressing
30BNL180	Boxed Sterile Large Wound Dressing
30BNL180-10	Sterile Large Wound Dressing (Pack of 10)
30BNM120	Boxed Unmedicated Sterile Medium Wound Dressing
145	Sterile Eye pad and Bandage Universal Sterile
30OEPNW1	Sterile Non-Woven Eye pad
100010	Sterile Medium Trauma Dressing
100011	Sterile Large Trauma Dressing
BP30BDFD35	Boots Sterile Medium Finger Bandage
1401020	Astroplast Flow Wrapped No16 Eye-pad & Bandage
1401001	Astroplast Medium Flow Wrapped Dressing 12cm x 12cm (Each)
1401002	Astroplast Large Flow Wrapped Dressing 18cm x 18cm (Each)
1401003	Astroplast Small No7 Flow Wrapped Finger Dressing 5cm x 5cm (Each)
1401018	Astroplast Extra-Large Flow Wrapped Dressing 28cm x 20cm (Each)
32502	Medikit Sterile No.16 Eye Pad Dressing
32504	Medikit Sterile Medium Dressing
32506	Medikit Sterile Large Dressing
77522	Medikit Sterile Boxed No.16 Eye Pad
83228	Medikit No.2 Ambulance Dressing
83244	Medikit Sterile Finger Bandage 3.5cm x 3.5cm

	Product Specification: TF-18	PS-0007-2020
	Sterile First Aid Dressings	
	Version: 1.03	Page 2 of 3

Device Description:

Sterile First Aid Dressing Bandage is applicable for haemostasis and isolating wound in first-aid.



Intended Use:

The sterile first-aid dressing bandage is applicable for haemostasis to trauma and isolating wounds in first-aid.

Contraindications: None

Precautions:

The following precautionary measures are advised;

- Do not touch the face of the sterile pad
- Sterile unless opened or damaged
- Single use only

Instructions For Use:

- To open the dressing, tear along serrated edge
- Do not touch the face of the sterile pad
- Place over wound and secure in place with bandage
- Re-use of product increases the risk of infection
- Sterile unless opened or damaged
- Store in a cool, dry place

Astroplast: Instructions for use are not necessary as it is a simple product.


Sterilisation: Gamma Radiation

Classification: Class I Sterile, according to Rule 4 Annex IX, MDD/93/42/EEC.


Sterile First Aid Dressings

Storage Conditions: Store in a cool dry place. Stored appropriately, the product has a shelf life of 5 years expiry in intact packaging.

Details of Test Specification and method Reference:

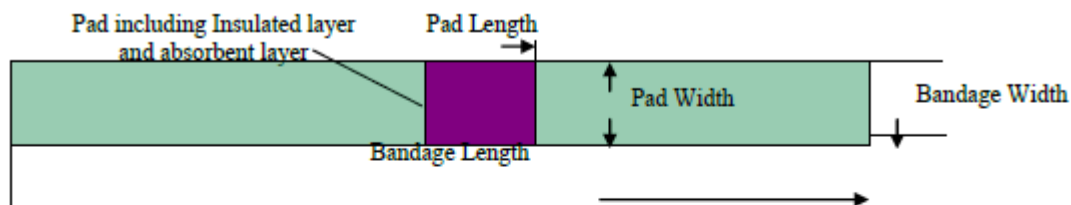

Sterile absorbent pad side elevation:

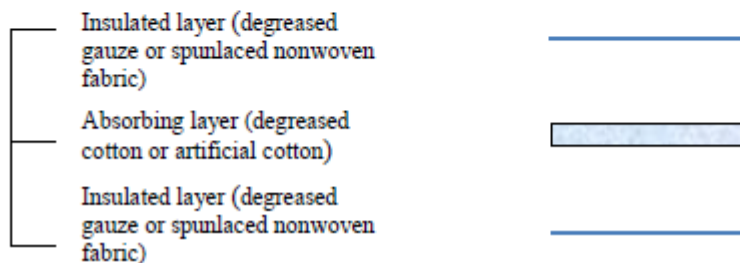

Items \ Model	13# First aid dressing bandage	14# First aid dressing bandage	15# First aid dressing bandage
Weight of degreased cotton (g/pc)	6±1	12±1	18±2
Insulated layer Length (±2cm)	20	28	35
Insulated layer Width (±0.1cm)	23	27	38
Absorbent Pad Length (±0.5cm)	10	15	20
Absorbent Pad Width (±0.5cm)	7.5	10	15
Bandage Length (±2cm)	200	250	350
Bandage Width (±0.1cm)	5	7.5	7.5
Final Sizes (±0.1cm)	5.5×3.5×2.5	8.2×4.8×2.5	8.2×5.2×2.7

Material Safety Data Sheet (MSDS) – none is required for these products.